

**Substances that elicit positive and false-positive results using
Immunalysis reagents**

Immunalysis Oxycodone Assay (100 ng cutoff) Calibrated with Oxycodone at 100 ng/ml

Drug	ng/ml
Oxycodone	100
Oxymorphone	100
Noroxymorphone	5,000
Oxymorphone-3 β -Glucuronide	500
Noroxycodone	7,500
Naloxone	3,750
Naloxone-3-Glucuronide	50,000
Naltrexone	30,000
Morphine	350,000
Normorphine	1,000,000
Codeine	500,000
Dihydrocodeine	100,000
Norcodeine	1,000,000
Heroin	300,000
Hydromorphone	50,000
Hydrocodone	100,000
Meperidine	50,000
Levophanol	200,000
Buprenorphine	10,000
Morphine-3 β -Glucuronide	500,000
6-Acetyl Morphine	100,000

Known False Positives: None